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EN Instructions for use
Product description
HydroClean plus is a hydroactive wound dressing that contains, as central component, a superabsorbent polyacrylate (SAP) embedded in cellulose fibres. The antibacterial agent polyhexamethylene biguanide (PHMB) is bound to the SAP that is activated with Ringer's solution. The wound contact layer consists of a polypropylene knitted fabric to which silicone strips have been applied, which additionally counteract adhesion of the dressing to the wound. On the side facing away from the wound, a water-impermeable polypropylene foil overlies with a hydrophobic polypropylene non-woven fabric prevents premature drying out of the wound dressing pad, so that moisture can be delivered to the wound for up to three days, while the foil at the same time largely prevents soaking of the outer layer of the dressing.

Properties and mode of action
HydroClean plus delivers Ringer's solution to the wound for up to three days. HydroClean plus continues to absorb liquid from the wound. The wound dressing absorbs wound exudate at the same time. The PHMB bound in the SAP core has an antibacterial effect and inhibits the growth of germs inside the wound cavity, thereby reducing the risk of recontamination over the entire application period of up to three days. The SAP used in HydroClean plus inactivates matrix metalloproteinases (MMP) which impair wound healing. As a result, a stagnating healing process in chronic wounds can be reactivated.

Indications
For wet treatment of wounds, particularly such with impaired healing tendency. HydroClean plus is especially suitable for the treatment of chronic and poorly healing wounds during the cleaning and granulation stage. HydroClean plus can furthermore be applied in order to reactivate stagnating healing processes if these are caused by excessive MMP activity.

Mode of application
HydroClean plus is placed on the wound with the white side of the dressing facing the wound and the blue inscription facing upwards. HydroClean plus must be in contact with the wound bed and should overlap the wound edges slightly. Deep wounds can additionally be packed with HydroClean plus cavity and can then be covered with HydroClean plus. HydroClean plus should be secured with suitable dressing retention material. The dressing must be changed when this is indicated from the medical point of view, but at the latest after three days. If the wound requires large amounts of moisture, the wound dressing can dry out prematurely and may need changing sooner. In most cases, HydroClean plus can be removed from the wound without any problems. Should the wound dressing pad adhere to the wound, the wound dressing can be rinsed underneath with physiological saline solution or Ringer's solution prior to removal and can then be removed almost painlessly after a few minutes.

Side effects
At the start of the treatment with HydroClean plus, breakdown of irreversibly damaged tissue can lead to an increase in the size of the wound. This can be a sign that wound healing has commenced. Reddening of the wound edges can occur during wet therapy; this is normally a sign that the blood circulation has been reactivated as the wound begins to heal. This is usually an indication that the wound is starting to heal. However, the cause must be assessed individually in each case. The HydroClean plus therapy as such does not cause any pain. However, the wound is highly sensitive to mechanical stress. The question whether an adequate analgesic therapy is indicated must be considered on a case-by-case basis. While wet therapy with Ringer's solution does not lead to maceration of vital cells, it can cause swelling of the soaked plantar horn layer, which then appears as a whitish coating.

Contraindications
Do not use HydroClean plus on patients who may be allergic to any of its ingredients.

Special precautions
HydroClean plus must not be cut to size or otherwise damaged mechanically. If the skin around the wound is particularly sensitive and impaired, it is advisable to protect the skin with a suitable skin protection preparation. In patients with increased bleeding tendency (e.g. due to blood-thinning medications), the wound dressings should be removed with particular caution when changing dressings. If necessary, the wound dressing should first be rinsed underneath with Ringer's solution or physiological saline solution as described above. Infected wounds and wounds highly exposed to bacteria should only be treated under the supervision of a physician, particularly if a film dressing is used for fixation. CAUTION: Before treating wounds with impaired healing tendency, the wound condition and the causes of the impairment of wound healing must always be assessed by a physician. Treatment with HydroClean plus cannot replace treatment of the cause of impaired wound healing. In particular, it cannot replace systemic antibiotic therapy.

Special instructions
Do not store above 30 °C. Store in a dry place. Contents sterile unless peel package is damaged. Keep out of reach of children. Date of revision of the text: 2015-05

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