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## **Instructions for use**

Sorbalgon is a calcium alginate non-woven wound dressing (or ribbon). In contact with sodium salts – as found in wound exudate – the alginate fibres become gelatinous creating a moist wound environment. This in turn supports the natural healing process.

## **Properties & Mode of Action**

The non-woven structure of the dressing combined with its physical and chemical properties result in a high absorbency – particularly important in the treatment of moderate to heavily exuding wounds. The material adapts remarkably to the contours of the wound preventing dessication. For the treatment of cavity wounds, Sorbalgon T ribbons are available.

## **Indications**

For all moderate to heavily exuding or bleeding wounds.

## **Application**

Cleanse the wound in accordance with standard nursing practice. Apply the dressing to the wound or plug the wound with a Sorbalgon T ribbon and secure with a suitable secondary dressing. In case of dry wounds Sorbalgon can be moistened with TenderWet Solution or Ringer's solution. Sorbalgon may be left in-situ for extended periods (up to several days), depending on the level of wound secretion. The dressing should be changed – once it has fully turned into a gel – in accordance with nursing standards. Should any fibres remain in the wound, these may be easily re-moved by a wound irrigation solution\* and are broken down naturally by the body.

## **Side effects**

None have been observed or reported.

## **Contraindications**

There are no known contraindications associated with the use of Sorbalgon on exuding wounds.

## **Special precautions**

Medical assessment of the condition of the wound and the causes of the delayed wound healing are necessary before treating wounds with impaired healing. Treatment with Sorbalgon cannot replace treatment of the cause of the impaired healing.

## **General instructions**

Store in cool and dry conditions. Contents sterile unless peel packs are damaged. Do not re-sterilize. Keep out of reach of children.

\* Ringer's solution, Ringer's rinsing solution without any additives or TenderWet solution.

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